



because clean matters.

EARMOLDS

Ear wax and perspiration soil can be safely and thoroughly cleaned from earmolds allowing better tonal quality with an Ultrasonic Cleaning System from L&R.

Simply-

1. Place earmold into basket and immerse into tank containing Earmold Cleaner Concentrate at recommended dilution (See product label).
2. Cavitate 2 - 3 minutes.
3. Rinse thoroughly and dry.

EQUIPMENT NEEDED:

Machines, Accessories*: Q90 or Q140 Ultrasonic Cleaning System with Timer, Optional Heater**, Basket, Auxiliary Pan.

Solutions: Earmold Cleaner Concentrate (Product Code 216)

*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.

