



because clean matters.

SURGICAL AND DENTAL INSTRUMENTS

Dried blood, tissue and protein, as well as common grit and grime are easily and safely cleaned with an L&R Ultrasonic Cleaning System and the proper cleaning solutions.

Simply –

1. Fill tank to 3/4 full with proper cleaning solution as per instructions on the container.
2. Place instruments into a solid side basket.
3. Place basket into machine and cavitate for 3 - 5 minutes.
4. Rinse thoroughly, proceed to next sterilization procedure and dry.

EQUIPMENT NEEDED:

Machines, Accessories*:
System with Timer, Basket.

Q140 or other Ultrasonic Cleaning

Solutions:

General Purpose Cleaner Non-ammoniated for Dental instruments (PC228); General Medical Cleaner Concentrate Non-ammoniated (PC149).

*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.

