



because clean matters.

TATTOOING NEEDLES -- HUMANS

Clean oxides and sulphides safely and quickly with L&R Ultrasonic Cleaning Systems and proper cleaning solutions.

Simply-

1. Fill Q140 with appropriate dilution of General Purpose Cleaner Concentrate, Ammoniated, to within one inch of top of tank.
2. Carefully place needle assembly in Basket. Do not allow needle tips to touch tank bottom.
3. Cavitate for 6 minutes.
4. Rinse thoroughly in water.
5. Dry and sterilize.

EQUIPMENT NEEDED:

Machines, Accessories*: Q140 or other Ultrasonic Cleaning System with Timer, Basket.

Solution: General Purpose Cleaner Concentrate, Ammoniated (PC226).

*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.

