



because clean matters.

VETERINARY HOSPITALS, ANIMAL CLINICS

Rather than hand cleaning and dulling sharp cutting edges and wasting time cleaning hard-to-get-at places in pipettes, currettes, scalers, trimmers and probes, modern 3-dimensional cleaning is available with L&R Ultrasonic Cleaning Systems.

Simply-

1. Place Basket containing instruments into tank filled with properly diluted General Medical Cleaner Concentrate (see label for dilution rates). Hinged instruments should be opened.
2. Cavitate 5 minutes.
3. Remove Basket. Rinse thoroughly in water.
4. Sterilize per normal procedure.

EQUIPMENT NEEDED:

Machines, Accessories*: Q140, Q210 or other Ultrasonic Cleaning System with Timer, Basket, Auxiliary Pan.

Solution: General Medical Cleaner Concentrate (PC149).

*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.

